

5mL Premium Selectable Injector

The Forward Selectable injector uses a positive stroke, that displaces the entire contents of the barrel at any dose setting reducing the chance of bubbles in the system. This ensures the dose is fully administered for dose accuracy and maximum efficacy. Ideal for administering 2 mL & 5 mL medications.


INCLUDES:

- ✔ 5mL Premium Injector
- ✔ 3 Vented Draw-off Caps (20mm, 27mm, & 30mm)
- ✔ 4ft Feed Tube
- ✔ Ergonomic Handle with Textured Non-slip Grip Finish

Specifications	
Product Number	850 0000-026
Injector Type	Tube
Dose Increments	.5mL
Dose Range	.5mL – 5mL
Dose Type	Forward Selectable
Species	Cattle, Swine, Sheep, Goat
Service Kit	Product Number: 850 0000-352 Kit Includes: 1-5mL Barrel; 5 O-rings



PREMIUM SELECTABLE INJECTOR WITH APPLICATOR BARREL

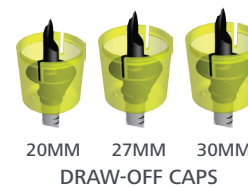

20MM 27MM 30MM
DRAW-OFF CAPS



TUBE



◀ Learn More

VISIT SIMCRO.COM

Customer Support **800-433-3112** or customerservice@datamars.com

.....
NJ Phillips is the American name for all Simcro branded products. NJ Phillips and Simcro are a registered trademarks of Datamars, Inc.
© 2024 Datamars, Inc. – All rights reserved. REV 12192023



How To Use Instructions

- 1** Compress the injector handle.
- 2** Turn the dose selector clockwise or counter-clockwise until the required dose appears in the dose window.
- 3** Once required dose is displayed in the dose window, let the injector handle open as normal.

The non-slip selector allows easy, accurate dose choice, without product loss when changing the dose setting, provided the handle is not released after previous dose administered. The indicator window clearly displays the dose volume. When smaller dose sizes are selected the plunger will move further forward along the barrel.

